

SARAYA

Skinaru

Science-Based Skincare for a Wide Range of Skin Types

Perineal Cleaning Wipes (Refreshing)

Effectively removes contaminants using thick wipes



Designed for single care,
using all five wipes

Can be
warmed



Fragrance-Free

Mildly
acidic



- 5 wipes per pack. Sheet size: 200 × 200 mm
- Dermatologically tested

* This doesn't mean that no one will experience skin irritation

Disposable wipes that simplify perineal care and support cost reduction and standardized procedures. Designed to reduce infection risks associated with water splashing and reuse of shower bottles.

Required items



This product



Gloves



Waste bag

No need

Reduce item and waste



Cloths, Gauze



Shower bottle



Absorbent pads



Aprons

Formulated with our proprietary natural cleansing ingredient, SOFORO*, to achieve both high cleansing performance and low skin irritation.



*Natural Ingredient: Sophorolipid

Sophorolipid is a naturally derived biosurfactant produced by yeast fermentation using plant oils and sugars. Made from renewable raw materials, it reduces CO₂ emissions during production and is biodegradable after use, minimizing environmental impact. It offers excellent cleansing performance with low skin irritation, making it suitable for hand soaps and healthcare hygiene products. This ingredient supports both safety and sustainability.


• Can be warmed in a towel warmer with the package opened.

When using a towel warmer, storage at 45°C for up to 8 hours is recommended. Prolonged storage may cause drying and should be avoided.

Technical Information

Moisturizing Effect

Method: Baseline stratum corneum hydration was measured on the forearm. Distilled water or product extract was then applied, and skin hydration was measured at defined time points.

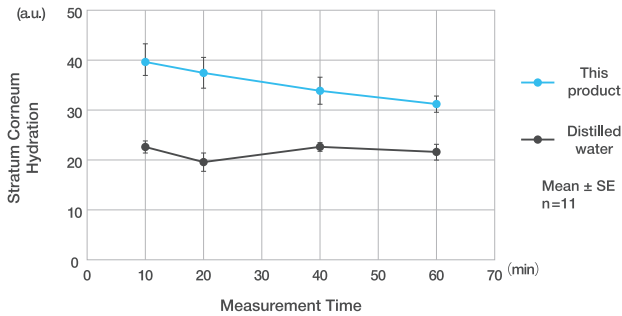

Result: Compared with competitor products and distilled water wipes, Skinaru Perineal Cleaning Wipes (Refreshing) showed a higher level of skin hydration after wiping.

Skin Irritation Test

Method: Twenty healthy adult volunteers participated. Two test samples were applied to the forearm for 20 minutes. After removal, the application sites were visually evaluated at defined intervals. Scoring was based on the Contact Dermatitis Clinical Practice Guidelines. The skin irritation index was calculated as: (Total score / Number of subjects) × 100.

Skin reaction	Grade	Score
No reaction	—	0
Slight erythema	±	0.5
Erythema	+	1.0
Erythema+ edema/papules	++	2.0
Erythema+ edema+ papules+ small vesicles	+++	3.0
Large vesicles	++++	4.0

Test material	Score	Skin irritation index	Classification
This product (1.5×1.5cm)	0	0	Safe
Control: Product cloth (1.5×1.5cm) soaked with physiological saline	0	0	Safe

Evaluation criteria based on Contact Dermatitis Clinical Practice Guidelines

Result: The skin irritation index was 0, confirming good skin compatibility. *Note: This doesn't mean that no one will experience skin irritation.*

Deodorizing Effect


Method: Aqueous solutions of two major urine odor components were added to the product extract and incubated. Residual odor compound concentrations were measured to calculate the deodorization rate.

Deodorizing effect against Ammonia			Deodorizing effect against Trimethylamine		
	Residual Ammonia Concentration (ppm)	Deodorization Rate (%)		Residual Trimethylamine Concentration (ppm)	Deodorization Rate (%)
Control	160.0	—	Control	21.0	—
This product	8.0	95.0	This product	<0.3	98.6<

Result: The product demonstrated effective deodorization against both urine odor components.

Fungicidal Wiping Efficacy


Method: A wiping test was conducted in accordance with JIS S 3303 test methods for evaluating the fungicidal performance of wet wipes.

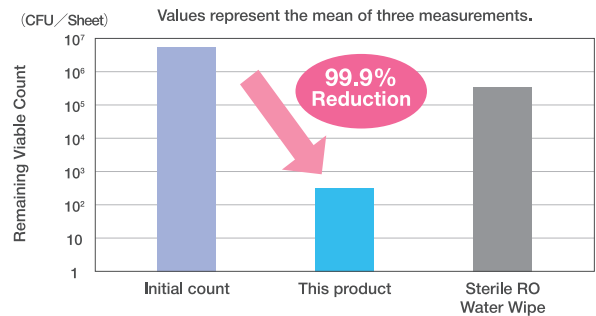

Result: Compared with sterile RO water wipes, this product demonstrated superior wiping efficacy against fungi.

Instruction for Use

Please wipe according to the steps below. Use one sheet per step. Do not allow the same surface of the wipe to contact previously cleaned areas.

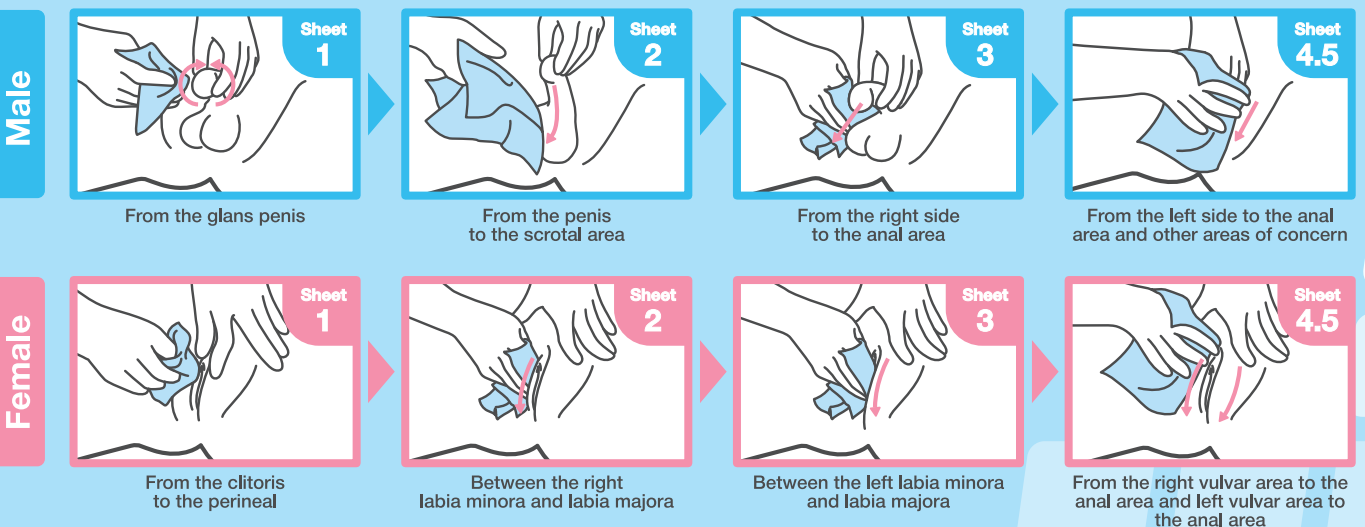

Product	Content	Units per Carton	Product Code	JAN Code
Skinaru Perineal Cleaning Wipes (Refreshing)	5 wipes x 10pcs	12 boxes	42370	49-87696-42370-1

- Product specifications may be changed without prior notice for improvement purposes.
- Please note that the actual product color may differ slightly from the images due to printing and photography conditions.
- The information provided is current as of December 2025.

SARAYA

2-2-8 Yuzato, Higashiumiyoshi-ku, Osaka 546-0013, Japan
 Phone: +81-6-6703-6336
 URL: <https://saraya.world/>